Fabius MRI

de	Ergänzung, Seite 2	tr	Ek, sayfa 36
en	Supplement, page 4	el	Παράρτημα, σελίδα 38
enUS	Supplement, page 6	ro	Supliment, pagina 40
fr	Supplément, page 8	ru	Дополнение, стр. 42
es	Suplemento, página 10	zh	使用说明增补文档, 第 44 页
it	Supplemento, pagina 12	ko	부록, 페이지 46
ptBR	Suplemento, página 14	ja	補足文書, ページ 48
nl	Aanvulling, pagina 16		
da	Tillæg, side 18		
no	Tillegg, side 20		
sv	Tillägg, sida 22		
cs	Dodatek, strana 24		
hr	Dodatak, stranica 26		
sr	Dodatak, strana 28		
sk	Doplňok, strana 30		
hu	Kiegészítés, oldal 32		
pl	Dodatek, strona 34		

WARNING

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Fabius MRI

Supplement to the instructions for use

Device	Part number	Edition
Fabius MRI	9039036	Up to 8. Edition

- Keep this supplement with the instructions for use of the medical device.



The supplement updates the information of the instructions for use in the following chapters:

Operating concept

LED indicators

There are several LED indicators on the front of the device.

LEDs for operating status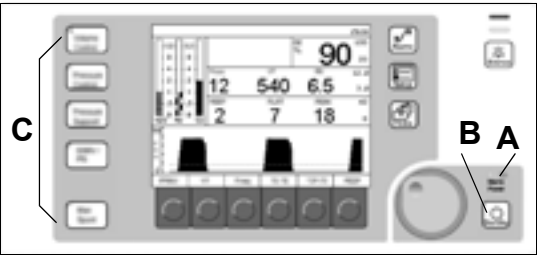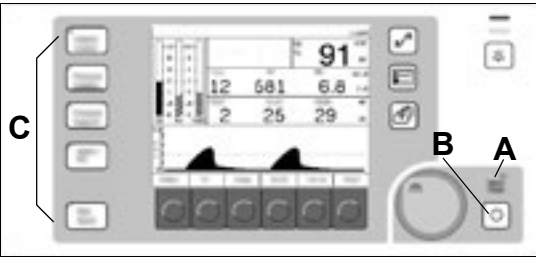


- If the device is connected to a mains power supply, the mains power LED (A) is lit up.
- In addition, the standby key (B) and all the keys for the ventilation modes (C) have small LEDs that light up when the respective mode is active.


LEDs for alarm status

Fabius MRI has alarm LEDs and alarm LED bars which signal alarms and indicate the alarm priority. Further information can be found in the "Alarms" chapter.

Alarm LEDs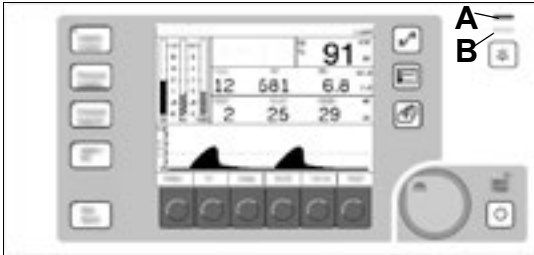


23476

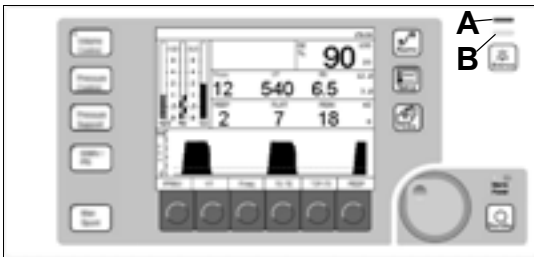23477

- Warning: Red LED (A) flashes
- Caution: Yellow LED (B) flashes
- Note: Yellow LED (B) lit constantly

Alarm LED bars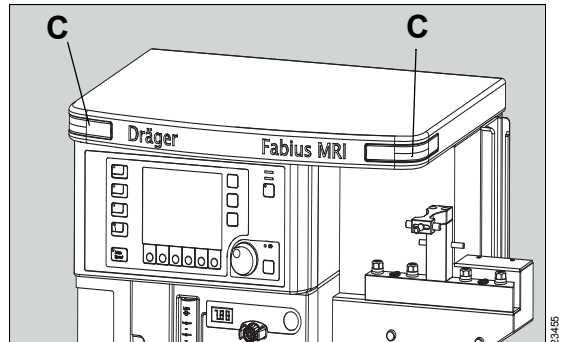

23485

- The alarm LED bars are triggered by the same causes as the alarm LEDs (A and B).

Signaling in the event of a device failure


CAUTION

Risk of alarms not being noticed

If the device is not positioned correctly, there is a risk that alarms may be noticed too late or not at all.

- Position the device so that the alarm LED bars are visible from various viewing angles and from every location.
- Do not cover the alarm LED bars.

Example: Left-hand side of the device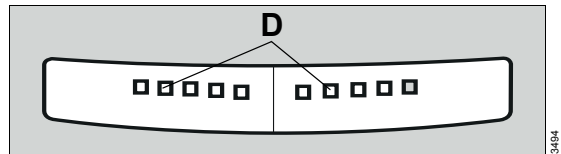

23484

- If the mains power supply and the battery fail, the device will no longer be ready for operation. Only the red LEDs (D) in the alarm LED bars (C) will be flashing.

Fabius MRI

Supplement to the instructions for use

Device	Part number	Edition
Fabius MRI	9039035	Up to 10. Edition

- Keep this supplement with the instructions for use of the medical device.



The supplement updates the information of the instructions for use in the following chapters:

Operating concept

LED indicators

There are several LED indicators on the front of the device.

LEDs for operating status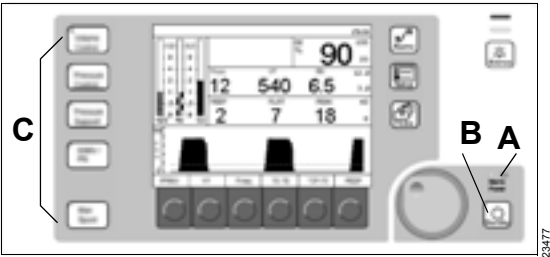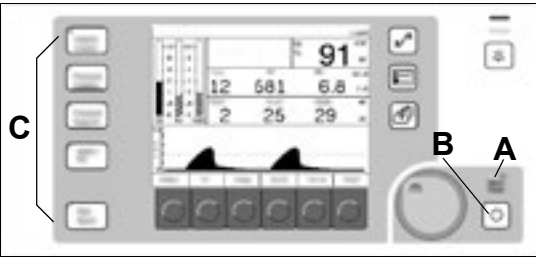


- If the device is connected to a mains power supply, the mains power LED (A) is lit up.
- In addition, the standby key (B) and all the keys for the ventilation modes (C) have small LEDs that light up when the respective mode is active.


LEDs for alarm status

Fabius MRI has alarm LEDs and alarm LED bars which signal alarms and indicate the alarm priority. Further information can be found in the "Alarms" chapter.

Alarm LEDs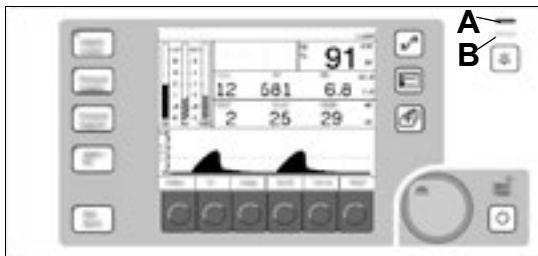


23476

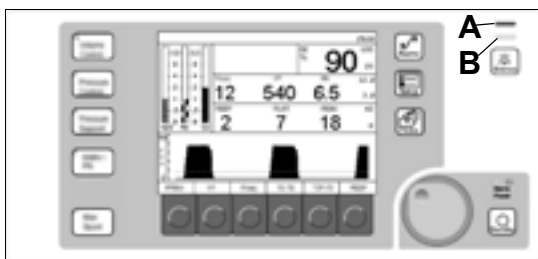23477

- Warning: Red LED (A) flashes
- Caution: Yellow LED (B) flashes
- Note: Yellow LED (B) lit constantly

Alarm LED bars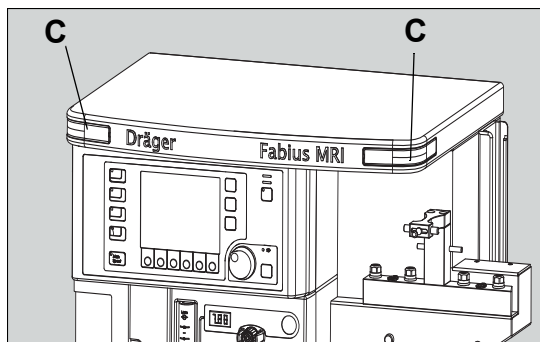

23485

- The alarm LED bars are triggered by the same causes as the alarm LEDs (A and B).

Signaling in the event of a device failure


CAUTION

Risk of alarms not being noticed

If the device is not positioned correctly, there is a risk that alarms may be noticed too late or not at all.

- Position the device so that the alarm LED bars are visible from various viewing angles and from every location.
- Do not cover the alarm LED bars.

Example: Left-hand side of the device

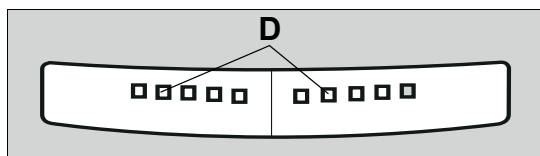23484

- If the mains power supply and the battery fail, the device will no longer be ready for operation. Only the red LEDs (D) in the alarm LED bars (C) will be flashing.

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© Dräger Medical GmbH
Dräger Medical GmbH
Draeger Medical Inc.
U.S.A.
Drägerwerk AG & Co. KGaA